

Analysis available

HLA-B*5701

Samples

Whole blood samples should be collected in 4ml EDTA tubes (plastic). **Do not send plasma.** The minimum blood sample volume is 1 ml. Please do not overfill tubes and allow for expansion if the sample has to be frozen (see below).

Sample storage

If collection and transport of the samples is to occur within 5 days then the samples should be kept at an ambient temperature prior to transport. If transport is to be later than 5 days after collection, blood samples should be stored between -20°C & -40°C (NOT -70°C), packed whilst frozen and allowed to thaw in transit.

Test request form

Please complete the sample request form and ensure that the samples are anonymised and that the full name of the patient does not appear on the form or the sample tube(s). Do not staple the form to sample bags as it presents a hazard in the laboratory. Ensure that the form is signed by the requesting physician. The form is available from our web-site at www.delphicdiagnostics.com

Transport

Studies have shown that samples may remain at room temperature for up to 6 weeks with no effect on DNA integrity, allowing overnight transportation without the need for packing on dry ice.

Results

Samples will be analysed as soon as possible and within 10 days of shipment. Results will be emailed to you (as per email address(es) on Account Set-up form).

Delivery address

Delphic Diagnostics
1030 Heeley Close
Kent Science Park
Sittingbourne
Kent ME9 8HL
UK

Contact details

For further information:

Tel: +44.20.7499.0777

Fax: +44.20.7499.0775

customerservices@delphicdiagnostics.com

www.delphicdiagnostics.com